



MAGNESIUM USER MANUAL



MagSHOCK®

BUILT OFF-ROAD TOUGH

— FAA & IATA Approved

We have designed our chairs to travel
Get out of your house and see the world
Do what your heart desires

Automobile ♦ Airplane ♦ Train ♦ Cruise Ship ♦ RV



Explore your World Drive Rain or Shine

Disabilities Don't take a Break for the Weather or
Rough Terrain, and neither should Your Mobility.

**Our 3-Year Warranty covers ANY & ALL
Weather Driving Conditions!**



WEATHERPROOF



DO NOT SUBMERGE THE WHEELCHAIR
INTO RIVERS, PONDS, LAKES,
BATHROOM SHOWERS, SWIMMING
POOLS OR THE OCEAN.

For The Adventurous Souls



Bump-Dampening

Our **Magnesium Oval Frame** is designed to absorb the bumps and shocks of everyday terrain. Since wheelchairs don't have traditional suspension, our frame does the work—giving you a smoother, more comfortable ride wherever you go.

Mountain Bike Frame

We took a page from the mountain biking world—where **magnesium frames** are prized for their strength, durability, and shock absorption. Just like mountain bikes our wheelchairs are built to tackle everyday terrain effortlessly.

The Ultimate Caregiver Remote Bluetooth for Wireless Driving

JOYSTICK KNOB: Use the control knob to steer the chair 360 degrees.

LOCK BUTTON: Press this button to **LOCK** the Chair's Joystick from driving, while this Remote is actively driving the chair.



POWER ON/OFF & UNLOCK: Press this button to **UNLOCK** the Chair's Joystick to enable driving and to turn off this Remote.

SPEED ADJUSTING TRIGGER: Press this Trigger to change Speed Gears from level 1-3.

AUTO SHUT OFF: This Remote will automatically power off when not used for more than **20 seconds**.

FOLD & GO WHEELCHAIRS

Speed adjusting

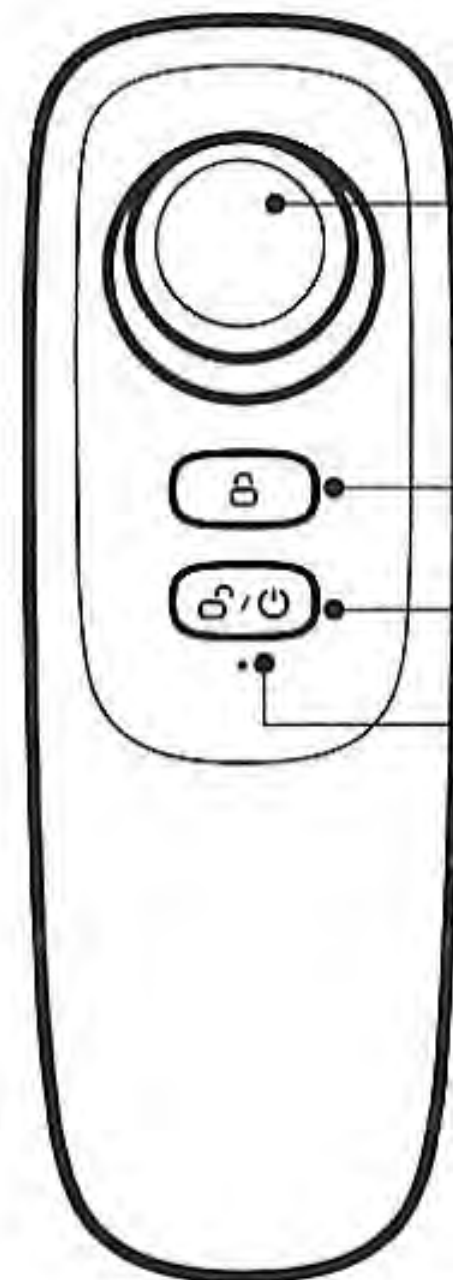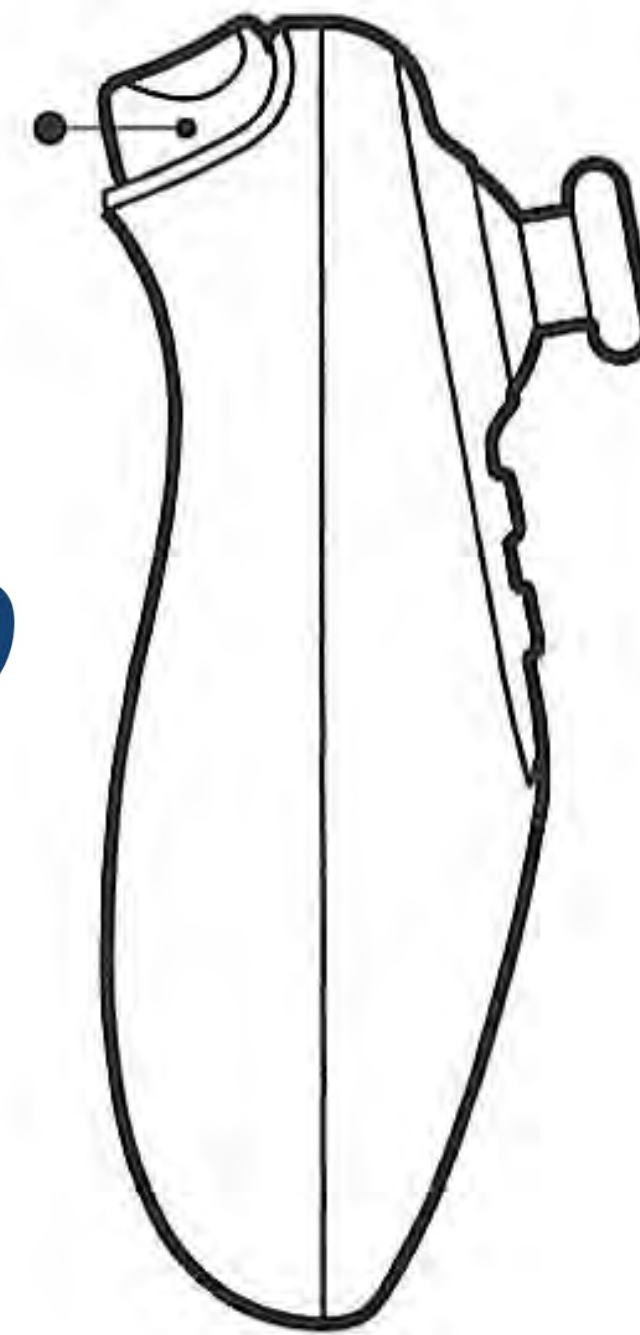

Joystick

Lock

Power on/ unlock

Indicating light



CAREGIVER REMOTE

Bluetooth Remote Control

Need a Chauffeur?

Every FOLD & GO comes with a Bluetooth Handheld Caregiver Remote Controller. Press the **Power Button** to turn ON/OFF. Press the **Lock Button** to prevent the Joystick from Driving the Chair.

When Using the Remote Control – Chair is limited to 50% Speed or Gear 3 for Safety.



HOW TO UNFOLD



Your FOLD & GO WHEELCHAIR is super simple to
unfold and unfold. It only takes 3-Steps.

Step 01

Gravity Assist

Hold the Backrest & Jerk to Open



Step 02

Frame Lock

Frame Locks Automatically when Unfolded



Step 03

Lower Armrests

Frame Automatically Locks when Unfolded



HOW TO FOLD



Your FOLD & GO WHEELCHAIR is super simple to fold and unfold. It only takes 3-Steps.

Step 01

Raise Armrests

Lift Up the Left and Right Armrests



Step 02

Frame Lock

Push the Lever to UNLOCK the Frame



Step 03


Gravity Assist

Push the Backrest Forward to Collapse




01 Drive Mode

Pull Back the RED Lever on Top of Each Motor Wheel, Towards the Rear of the Wheelchair

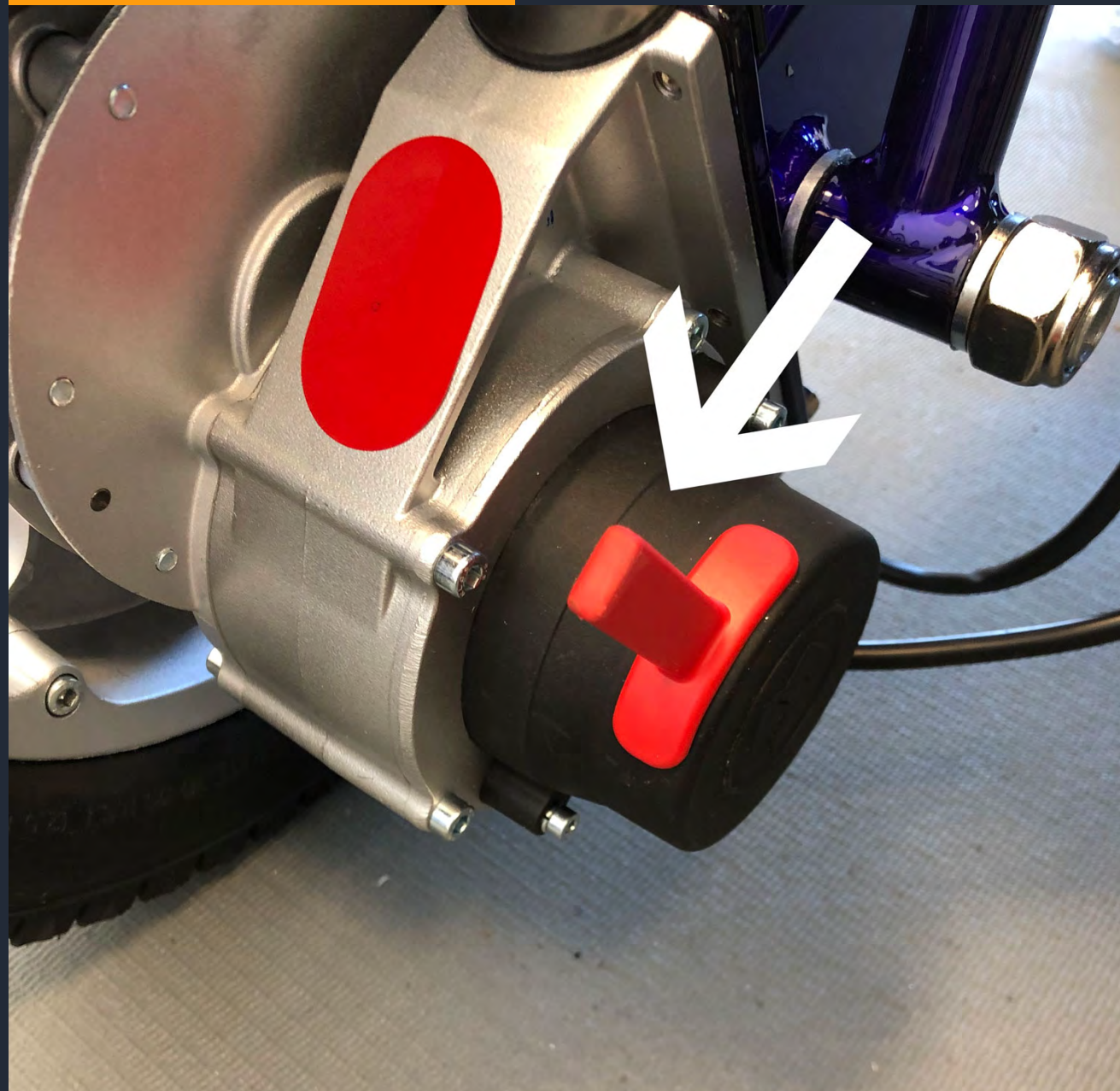

02 Manual Mode

Push forward the RED Lever on Top of Each Motor Wheel, Towards the Front of the Wheelchair

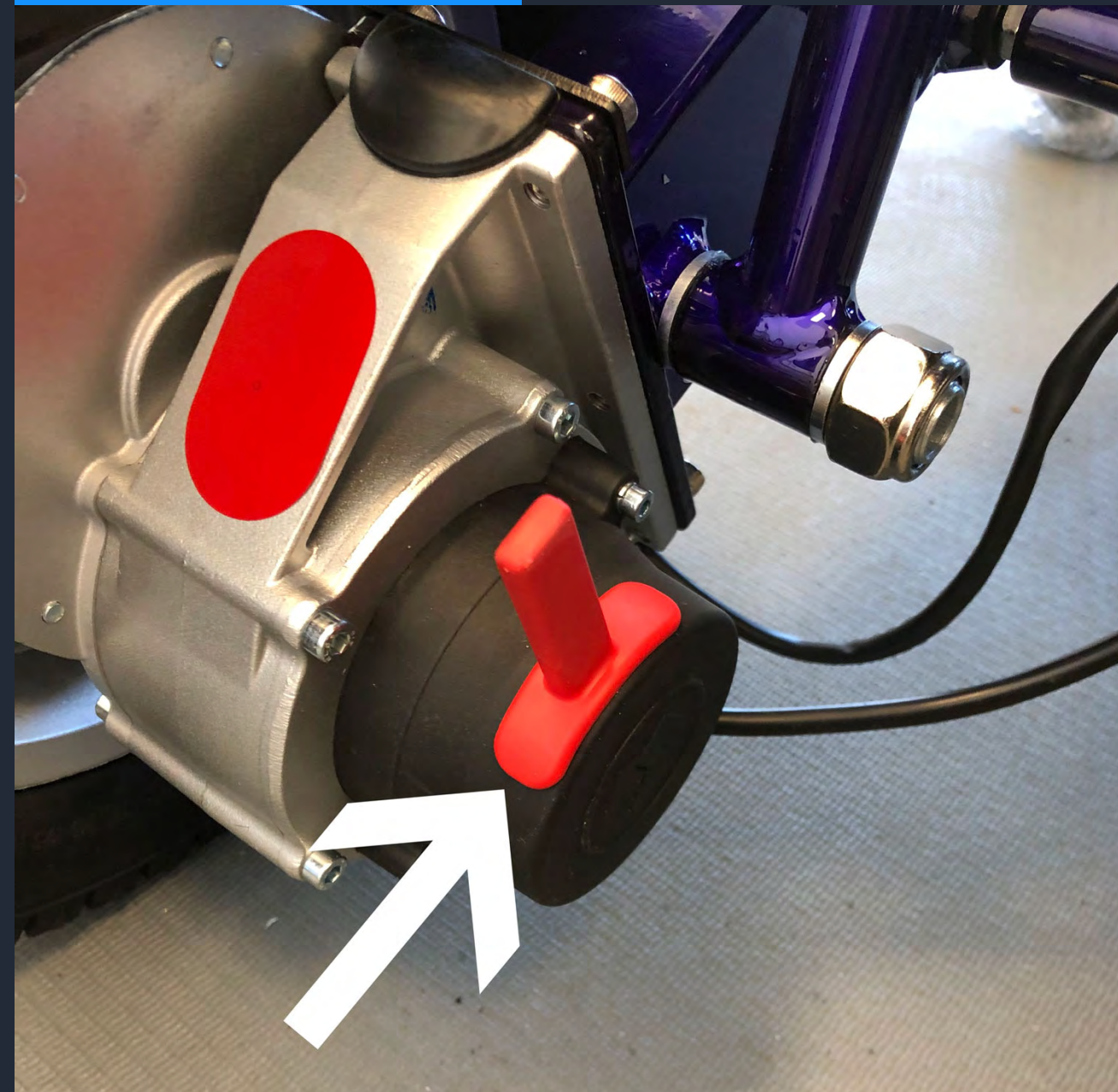

To **DRIVE** or To Be **PUSHED**

TO DRIVE: Your **FOLD & GO** is built for independence. Drive yourself wherever life takes you, with ease and confidence.

TO BE PUSHED: On those rare occasions when you need assistance, simply engage the **red levers** on each motor wheel, and your wheelchair is ready to be pushed.

PUSH LIKE A WALKER: Need to stretch your legs? No need for a separate cane or walker—just grab the bar above the backrest and push your **FOLD & GO** like a shopping cart. Stay active, your way!

INDEPENDENCE



Our Magnesium Frame curved to match your spine

-
-
-
-



Ergonomically Sculpted for All-Day Comfort

Our **Magnesium Frame** is designed with your body in mind. The **curved backrest frame** follows the natural shape of your spine, providing superior ergonomic support. Paired with our **contoured backrest and seat bottom cushions**, these carefully crafted curves ensure maximum comfort—so you can sit with ease, all day long.




The Advantages of MagCONTOUR™

Oval Frame Shape: Engineered for a Smoother Ride

Our **Oval Magnesium Frame** absorbs shocks and vibrations, ensuring a smoother ride over bricks, cobblestones, pavers, and rough terrain for maximum comfort and control.

Built-In: Seat Cushion Suspension

Two **Red Spring Shocks** beneath the contoured seat cushion absorb impact, adding extra comfort and support over rocky roads and uneven terrain.

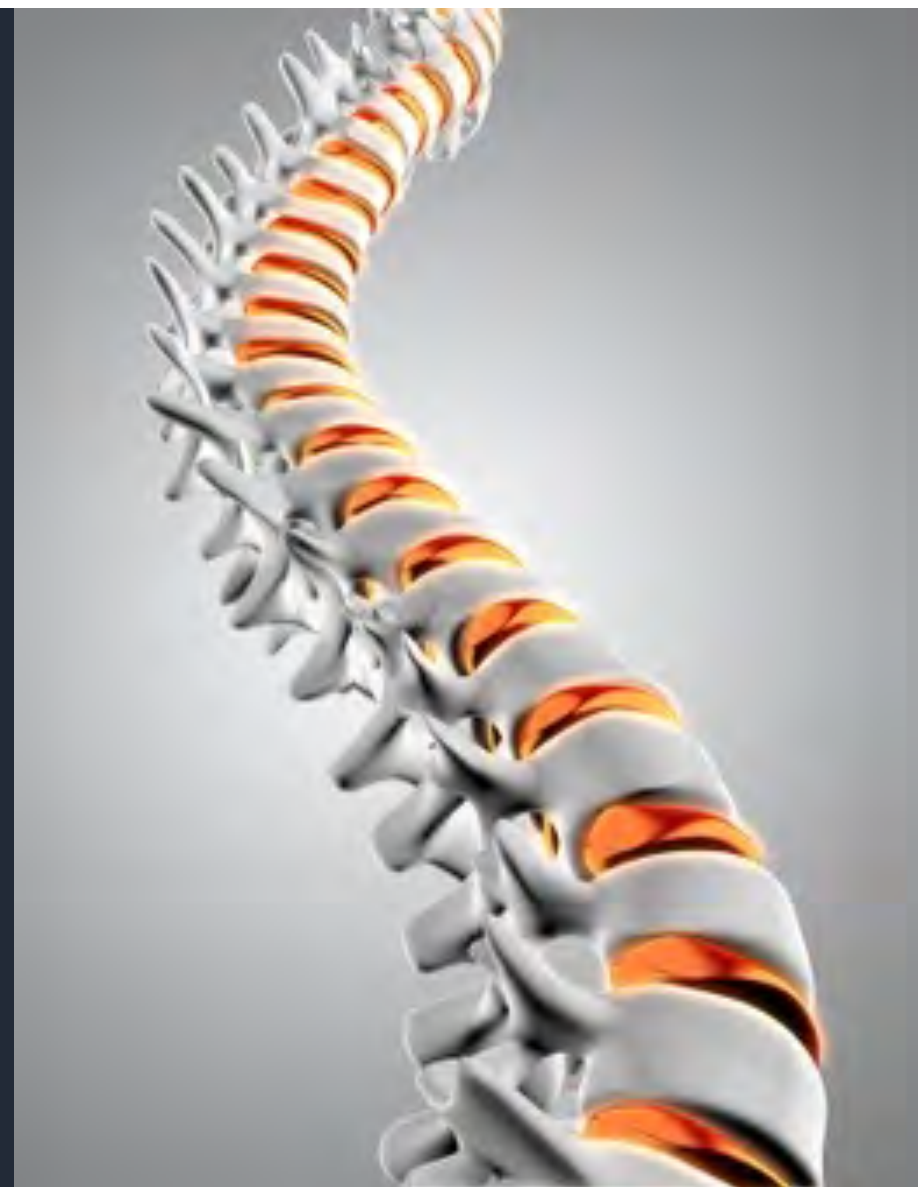

5 Gears Speed Control

Increase SPEED

Press the PLUS + button to increase the speed

Decrease SPEED

Press the MINUS - button to decrease the speed

5 Gears Speed & Torque

Gear & Torque Limits

Gear 1 = 0.5 MPH & 20% Torque

Gear 2 = 1 MPH & 40% Torque

Gear 3 = 2 MPH & 60% Torque

Gear 4 = 3 MPH & 80% Torque

Gear 5 = 4 MPH & 100% Torque



Power ON/OFF Button

Power ON/OFF

Press the Center button to turn ON/OFF the Wheelchair

Odometer

Keep Track of Every Mile in the Lower Right Corner

Battery Indicator Lights

Charging Bar Levels

5 White Bars Reflect the Battery Level. The White Bars Turn Off 1-by-1 Starting on the Right Side Down to the Last Bar.

BATTERY SAVER MODE: 1 Yellow Bar Remaining Chair Slows to 1 MPH Max speed to help Conserve Battery Life.

JOYSTICK



Press and hold the  and  buttons together for 7 seconds until it shows “”



How to **LOCK**
the Joystick



Press and hold the  and  buttons together for 7 seconds until it shows “”



How to **UNLOCK**
the Joystick

ERROR CODES

Code	Electrical Component	Troubleshooting
1	Joystick	Electrical System Startup
2	Motorwheel Red Levers	Chair is in Manual/Push Mode
12	Joystick	QuickConnect not plugged in
14	Joystick	Contact Our Service Department
22	Batteries	Contact Our Service Department
23	Charger	Extreme Cold Temperature
24	Computer (CPU)	Extreme Cold Temperature
25	Charger	Extreme Heat Temperature
26	Charger	Might Need a New Charger
27	Batteries	Extreme Heat Temperature
28	Batteries	Might Need New Batteries
29	Batteries	Using Wrong Battery Charger
30	Batteries	Batteries in Trickle Mode
31	Charger	Need New Charger
42	Joystick	QuickConnect not plugged in
43	Left Motorwheel	Unplugged from CPU
44	Left Motorwheel	Extreme Heat Temperature
45	Left Motorwheel	Contact Our Service Department
46	Left Motorwheel	Contact Our Service Department
47	Left Motorwheel	Motor Unplugged from CPU

Code	Electrical Component	Troubleshooting
48	Left Motorwheel EMV	Brakes Not Working
49	Left Motorwheel	Contact Our Service Department
50	Left Motorwheel	Contact Service Department
51	Left Motorwheel EMV	Brakes Not Working
52	Left Motorwheel EMV	Brakes Not Working
53	Left Motorwheel	Contact Our Service Department
54	Right Motorwheel	Contact Our Service Department
55	Right Motorwheel	Extreme Heat Temperature
56	Right Motorwheel	Contact Our Service Department
57	Right Motorwheel EMV	Brakes Not Working
58	Right Motorwheel	Motor Unplugged from CPU
59	Right Motorwheel	Contact Our Service Department
60	Right Motorwheel	Contact Our Service Department
61	Right Motorwheel	Contact Our Service Department
62	Right Motorwheel	Contact Our Service Department
63	Right Motorwheel	Contact Our Service Department
64	Right Motorwheel	Contact Our Service Department
65	Joystick	Contact Our Service Department
66	Joystick	Extreme Heat Temperature
72-99	Computer (CPU)	Contact Our Service Department

Contact Our In-House Service Department

We are here if you have any Errors that you can not troubleshoot by this table.

Please visit the **SERVICE Page** on our Website to Schedule a FREE ZOOM Video Appointment with one of our Certified Repair Technicians.



HOW TO CHARGE THE BATTERIES

ELECTRICITY CAUTION: Only plug the charger into a standard in-wall 110V or 220V outlet. Do not use a GFI outlet, light switch/plug combo, extension cord, power strip, metal external outlet or surge protector.



SINGLE Battery Charging

PLUG the charger **directly** into the port on top of each **battery**. This allows you to leave the chair in the car and charge the batteries separately indoors.

Total Charging Time: 10 hours—each battery takes 5 hours individually to fully charge.

TWO Charging Options



DUAL Battery Charging

PLUG the charger **directly** into the port on the bottom of the **joystick** to charge both batteries simultaneously.

Total Charging Time: Only 5 hours—the fastest way to power up your FOLD & GO.

16 MILES PER

CHARGE

BATTERY LIFE: 1000 CHARGES

MagSHOCK® Dimensions

- Net Weight with Batteries: 60 LBS
- Weight Capacity: 400 LBS
- Unfolded Height 38 inches
- Overall Width: 24 inches
- Folded Height 36 inches (with Joystick, 30 inches w/o)
- Folded Depth: 16 inches
- Seat Width Between Armrests: 20 inches
- Seat Depth: 20 inches
- Backrest Height: 21 inches
- Floor to Seat Cushion Height: 20 inches
- Floor to Armrest Height: 28 inches
- Rear All-Terrain Tread Wheel: 12 inches by 2.5 inches
- Front All-Terrain Tread Wheel: 8 inches by 2 inches
- Front to Back Wheelbase Length: 40 inches
- Super Tight Turning Radius: 32 inches



NO LIMITATIONS



25° HILL + CLIMBING

**More than Standard ADA 8-12° Slopes
Our Chairs Are Built to Climb the Great Outdoors**

From rugged trails to steep driveways, parking garages to cruise ship gangways, crosswalks to airport terminals—real-life obstacles demand more than just a standard wheelchair. That's why we designed our chairs to tackle it all, giving you the freedom to go where you want, with confidence.

NO BOUNDARIES

Common Sense Driving Practices

UNDERSTANDING YOUR ENVIRONMENT

Don't try to drive your FOLD & GO WHEELCHAIR while you are on a transportation vehicle, like a bus, subway, train, or airplane.

While using public transportation vehicles, ALWAYS hook the floor straps to the frame of your FOLD & GO WHEELCHAIR.

In many countries, power wheelchairs are NOT allowed to drive on public roads. You must observe and obey all traffic laws.

Your FOLD & GO WHEELCHAIR is designed to be weatherproof. Disassembling any of the electronic components or removing the batteries from their watertight housing sleeve will compromise the weatherproof seal on the electronics.

When driving your FOLD & GO WHEELCHAIR always sit back in the seat, avoid leaning or bending forward over the footrest.

Use caution driving your FOLD & GO WHEELCHAIR at night. Your chair comes with reflective stickers on the front and rear. Headlights and Taillights are also available under the accessories page on our website.



When driving your FOLD & GO WHEELCHAIR on slopes or hills, always approach at a straight angle to the incline. Choose the approach that best minimizes any side angles. If you do not make it up the slope on the first attempt reverse back down the slope while keeping the wheelchair straight. NEVER try to turn around while on a slope.

When getting in or out of your FOLD & GO WHEELCHAIR remember to turn OFF the joystick. Otherwise, if you bump the joystick with your arm, the chair will move unexpectedly on you.

Never attempt to drive your FOLD & GO WHEELCHAIR down or up a staircase, or escalator. No matter how much fun it looks like it would be.

When an elevator is not available, then fold up your FOLD & GO WHEELCHAIR and roll it up or down a staircase. Put the Motor wheels into MANUAL mode to help the chair roll up each step with ease.

MONTHLY MAINTENANCE

Your FOLD & GO Wheelchair is held together by roughly 2 Dozen Bolts. The same bolts that let it FOLD & GO so easily. To Keep your Chair driving straight and feeling solid, give every bolt a quick tightening once a month.

It takes about 5 minutes. Those 5 minutes add Years of life to your wheelchair.



Help Your Chair Last for YEARS TIGHTEN THE BOLTS

Monthly Email Reminders

Each month we will send you an email with a link to our YouTube Video showing exactly How to Tighten Your Bolts.

Tool Kit & Wrench Included

Every FOLD & GO ships with the Tool Kit & Wrench you need. Keep them with your chair, bolts can loosen on rough trails, on long drives, and especially during turbulence on flights.

Huh. Turns out
Monthly Maintenance
wasn't a suggestion.



WeatherPROOF

Our **Magnesium Wheelchairs** are built for **everyday use**—rain or shine. **Fully weatherproof**, they're designed to handle any conditions, so you can drive with confidence anywhere, anytime.

Our **3-Year Warranty** covers **ALL** weather driving conditions...**PERIOD!**



IMPORTANT WARNING
RINSE OFF ALL CHEMICALS:
After using your wheelchair at **water parks, beaches, lakes, ponds, or the ocean**, **immediately rinse it with clean water**. This prevents corrosion by removing harmful chemicals, saltwater, and debris from the metal frame, electrical system, and cables.



4-SEASONS of DRIVING CONDITIONS